


Helping You Wear it Right

3M™ Health Care Particulate Respirator and Surgical Masks, 1860/1860S

Application

Always read and follow User Instructions.

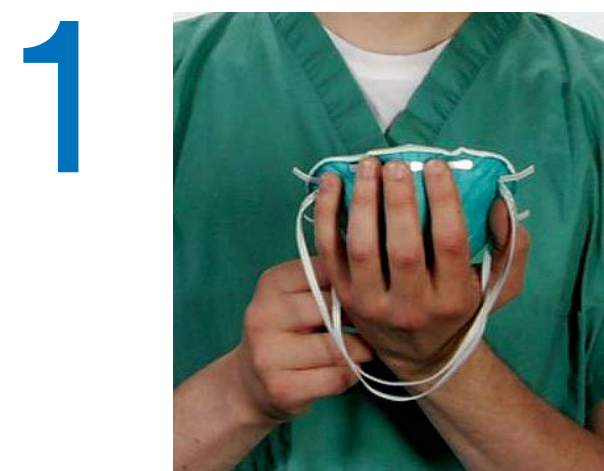


Cup the respirator in your hand with the nose piece at fingertips, allowing the head straps to hang freely below hand.



Position the respirator under your chin with the nose piece up.



While holding the respirator in place, pull the top strap over your head so it rests high on the back of your head.

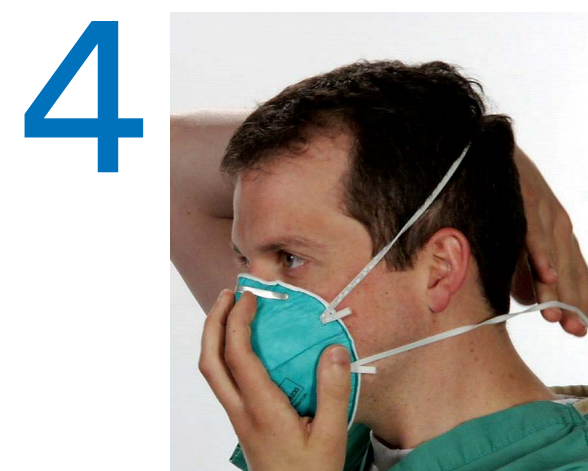

While continuing to hold the respirator firmly in place, pull the bottom strap over your head and position it around your neck, below your ears. Untwist the straps. Position the respirator low on your nose.



Using both hands, mold the nose piece to the shape of your nose by pushing inward while moving your fingertips down both sides of the nose piece.

Note: Always use two hands when molding nose piece. Pinching with one hand may result in improper fit and less effective respirator performance.




Perform a User Seal Check

Check the seal of your respirator each time you use the respirator.

To perform the user seal check, place both hands completely over the respirator, being careful not to disturb the position, and exhale sharply. If air leaks around your nose, adjust the nose piece as described in step 5. If air leaks at respirator edges, adjust the straps back along the sides of your head. Perform seal check again if an adjustment is made. **If you cannot achieve a proper fit, see your supervisor. Do not enter area requiring respirator use.**

Removal

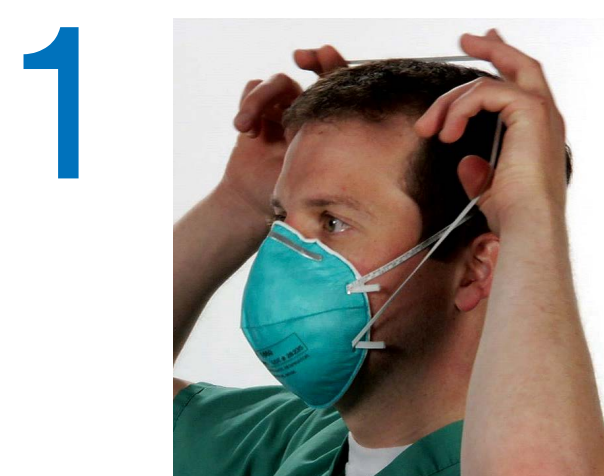

Without touching the respirator, slowly lift the bottom strap from around your neck up and over your head.



Lift off the top strap. Do not touch the respirator.



Store or discard according to your facility's infection control policy.



WARNING

Respirators help protect against certain airborne contaminants. Before use, the wearer must read and understand the *User Instructions* provided as a part of the product packaging. Follow all local regulations. In the U.S., a written respiratory protection program must be implemented meeting all the requirements of OSHA 29 CFR 1910.134 including training, fit testing and medical evaluation. In Canada, CSA standards Z94.4 requirements must be met and/or requirements of the applicable jurisdiction, as appropriate. **Misuse may result in sickness or death.** For correct use, consult supervisor and *User Instructions*, or call 3M Personal Safety Division (PSD) Technical Service in the U.S.A. at 1-800-243-4630. In Canada, call 1-800-267-4414.